



Programação dos Reagentes CEPA

Artax

ALT-GPT Cepa

Cinético UV-U/L - 37°C (IFCC)

PARAMETERS	ALT
METHOD	KINETIC
FILTER 1	340
FILTER 2	0
CUVETTE TEMP.	37
MAX. NORM. VAL.	41
MIN. NORM. VAL.	0
UNIT	U/L
STANDARD VAL.	0
FACTOR	3333
# OF WASHES	2..
KIN. READ TIME	30
PRINTOUT GRAPH	N
TEST LIMIT	500
MAX. ABS. DELTA	150
REAGENT (A) µL	500
REAGENT (B) µL	0
SAMP. VOL. µL	25
INC. REAG. (A)	60
INC. REAG. (B)	0
INC. TEMP.	37
STABILITY TIME	0
REACTION DIR.	DEC.