



## Programação dos Reagentes CEPA

RA-XT 500 - 1000

### ALT-GPT Ceba

Cinético UV-U/L - 37°C (IFCC)

|                 |            |
|-----------------|------------|
| CHEM #          | *          |
| NAME            | ALT/GPT    |
| IMMUNOASSAY     | NO         |
| TYPE            | ZERO ORDER |
| INVERSE         | YES        |
| % SAMPLE        | 30(15µL)   |
| FILTER P        | 340        |
| BIC. CHEM       | NO         |
| DEPLETION LIMIT | 0,2        |
| DELAY TIME      | 02:15:00   |
| BLANK           | NO         |
| %REAG. VOL      | 300µL      |
| 2° REAG.        | NO         |
| UNITS           | U/L        |
| UNIT FACTOR     | 1          |
| DECIMAL         | 0          |
| RBL LOW         | 0,7        |
| RBL HIGH        | 1400       |
| RANGE LOW       | 0          |
| RANGE HIGH      | 500        |
| CAL FACTOR (**) | 4762       |
| REG. RATE       | 0          |
| NORMAL LOW      | 0          |
| NORMAL HIGH     | 41         |
| SLOPE           | 1          |
| INTERCEPTION    | 0          |
| C1 *            | 0          |
| C2 *            | 99999      |
| D1 *            | 10         |
| DELTA#          | 0.01       |