

ALT-GPT Cepa

Cinético UV – U/L - 37°(IFCC)

TEST TYPE	KINETIC SINGLE
SERUM STARTER	INACTIVE
FILTERS A/B	340/700
UNITS	U/L
TEST METHOD	WHIT FACTOR
TEST METHODOLOGY	
NUMBER OF WASHES	1
DELAY TIME	0
INC REAGENT	120
READING TIME	120
TEST LIMIT (CONC)	500
MAX ABS DELTA	300
REAGENT mABS Limit	900
REAGENT A/B	300
REACTION DIRECTION	Decreasing
REAGENT DILUTION	01:01:00
INITIAL ABS	1999
CURVE ACCEPTANCE	100,00%
AUTOMATIC PROFILE	(INACTIVE)
RERUN TEST RGT BLK H:M	00:00:00
SERUM PARAMETERS	
TEST'S NAMES	ALT/TGP
SAMPLE VOLUME	30
DILUTION RATIO	1:5
MIN. MAX. M	0 – 41
MIN. MAX. F	0 – 41
MIN. MAX. B	