



Programação dos Reagentes CEPA

EXPRESS 550 PLUS

ASLO Turb. Cepa

Turbidimetria

Test name: ASLO	Test: ASLO
Test Bar Code: *	
Test Type: End Point	Curve Type: Blanked Linear
Units: IU/mL	No. of Decimal Places: 0
Primary Wavelength: 540	Secondary Wavelength: -
Read Time/Interval: 20	Sample Blank: No
Factor:	
Calibration Interval: *	
Normalization Interval: *	
No. of Calibrators: 2	No. of Replicates: 2
Low Blank A Limit: -0.100	High Blank A Limit: 2.000
Low A Limit: -0.100	High A Limit: 2.500
Low Normal: 0	High Normal: 200
Linearity Limit: 600	Curve S.D. Limit: 8.0
Test Name: ASLO	Test: ASLO
Test Bar Code: *	
Sample Volume: 3	Sample Diluent:
Rerun Dilution Ratio: 1	Predilution Ratio: 1
Reagent Diluent:	
	Reagent Bar Diluent Lag
	Volume Code Volume Time
Reagent 1	300 AS1A 120
Controls: *	