



Programação dos Reagentes CEPA SELECTRA II ®

ASLO Cepa Turbidimetria

Name	ASLO	Prozone check	No
Abbr. Name	*		
Mode	Two Point	Ref. male low	0 IU/L
		Ref. male high	200IU/L
Wavelength	546 nm	Ref. female low	0 IU/L
Units	IU/L	Ref. female high	200 IU/L
Decimals	0	Ref. ped. low	
Low Conc.	0 IU/L	Ref. ped. high	
High Conc.	800 U/L	Control 1	*
Calibrator Name			
Repeat	3	Control 2	*
Number	1		
Concentration		Control 3	*
Interval	*		
Cut-off	No	Correlat. factor	1.000
		Correlat. offset	0.000
MONO MODE			
Name	ASLO		
Sample blank	No	Low Absorbance	-0.050
R1 bottle	25 mL	High Absorbance	3.000
normal volume	300 µL	R. Abs. L. Limit	-0.050
rerun volume	300 µL	R. Abs. H. Limit	0.900
Sample		R.Abs.Deviation	0.150
normal volume	3 µL	Reagent Blank	Yes §
rerun volume	2 µL	Cal Low Limit	*
		Cal. High Limit	*
		Factor	*
Point one, two	12, 110 sec		