



ASLO CEPA

TURBIDIMETRIA

R1 – Tampão
R3 - Látex

ANALYTICAL PARAMETERS

Analytical Conditions		Reanalysis conditions				Multi-Standards Setting			
R1 volume	80	Serum reac. Smp. Vol (µL)			0	Formula Points	Linear correc. 2	Axis conv.	No converte
R2 volume	0	Serum dilut. method. Vol (µL)			None				
R3 volume	20	Serum reac. Smp. Vol (d)			0		FV	MEAN	
R4 volume	0	Serum dilut. method. Vol (d)			None	BLK	0.00	...	
R1 diluent volume	0	Standards setting				1	*	---	
R2 diluent volume	0	BLK H			9.9999				
R3 diluent volume	0	BLK L			9.9999				
R4 diluent volume	0	STD H			9.9999				
Serum reac. s.volume	3,5	STD L			9.9999				
Serum diluent method	Standard	FV			9.9999				
		Abnml. (serum) H			9.9999				
Reaction Time	10 min.	Abnml. (serum) L			9.9999				
Reagent 1 stir.	Weak	Calculation method settings				Reation rate method			
Reagent 2 stir.	Weak	M-DET. P.l	50	Prozone		Cycle	3		
Reagent 3 stir.	Weak	M-DET. P.m	68	Prozone form	None	Factor	3.0		
Reagent 4 stir.	Weak	M-DET. P.n	70	Prozone limit	9999	Reac. Type	INC		
Sub-analy. conditions		M-DET. P.p	0	Prozone judge	Upper Limit	E2. corre.	Not dot		
Name	ASLO	M-DET. P.r	0	Judge limit	9999	Blank (µL)	9.9999		
Digits	2			M-DET.P.m	0	Blank (d)	-9.999		
M-wave. L	545	Check D.P.l	0	M-DET.P.n	0	Sample (µL)	9.9999		
S-wave. L.	-	Limit value	0.003	S-DET. P.p	0	Sample (d)	-9.999		
Analy. mthd	EPA	Variance	10	S-DET.P.r	0	Endpoint method			
Calc. mtdh	MSTD					Re. Absorb (µL)	9.9999		
Qualit. judg	Not do					Re. Absorb (d)	-9.999		