



Programação dos Reagentes CEPA

COBAS MIRA / MIRA PLUS

Albumina Ceba

Verde de Bromocresol - Colorimétrico

GENERAL		CALCULATION	
MEASUREMENT MODE	ABSORB	SAMPLE LIMIT	NO
REACTION MODE	R-S	REACTION DIRECTION	INCREASE
CALIBRATION MODE	CALIBRATOR	CHECK	ON
REAGENT BLANK	REAG/DIL	CONVERSION FACTOR	1
CLEANER	NO	OFF SET	0
WAVELENGTH	600	TEST RANGE LOW	0 g/L
DECIMAL POSITION	1 g/L	HIGH	6 g/L
UNIT	7 g/L	NORMAL RANGE LOW	3,5 g/L
		HIGH	5,5 g/L
ANALYSIS		NUMBER OF STEPS	1
POST DIL FACTOR	NO	CALCULATION STEP A	ENDPOINT
CONC. FACTOR	NO	READING FIRST	CB
SAMPLE CYCLE	1	LAST	5
VOLUME	3 µL	REACTION LIMIT	-
DILUTION NAME	0	CALIBRATION	
VOLUME	20 µL	CALIBRATION INTERVAL	ON REQUEST
REAGENT CYCLE	1	TIME	NO
VOLUME	300 µL	BLANK RANGE LOW	-0,020
		HIGH	0,220
STANDARD POS	*	REAGENTE RANGE LOW	NO
STD 1	*	REAGENT RANGE HIGH	NO

*Inserido pelo usuário

Para Cobas Mira Plus, substituir CALIBRATOR por SLOPE AVG