



Programação dos Reagentes CEPA

Artax

Alfa Amilase Cepa

GAL G2 CNP – Cinético

PARAMETERS	AA
METHOD	KINETIC
FILTER 1	405
FILTER 2	0
CUVETTE TEMP.	37
MAX. NORM. VAL.	100
MIN. NORM. VAL.	28
UNIT	U/L
STANDARD VAL.	0
FACTOR	3239
# OF WASHES	2
KIN. READ TIME	60
PRINTOUT GRAPH	N
TEST LIMIT	1300
MAX. ABS. DELTA	280
REAGENT (A) μ L	500
REAGENT (B) μ L	0
SAMP. VOL. μ L	10
INC. REAG. (A)	60
INC. REAG. (B)	0
INC. TEMP.	37
STABILITY TIME	30
REACTION DIR.	INC. B