



Programação dos Reagentes CEPA

AIRONE 200®

Alfa Amilase Cepa

GAL G2 CNP – Cinético

TEST NAME	AMIL
DECIMAL	0
MEASURE UNIT	U/L
REACTION TYPE	KINETIC
SAMPLE VOLUME (□L)	12 (Soro e Plasma) / 7 (Urina)
REAGENT VOLUME (mL)	
REAGENT 1	400
REAGENT 2	0
ABSORBANCE RANGE (mABS)	
MINIMUM	0
MAXIMUM	2400
LINEARITY LIMIT	1300
CONTAMINANTING	NO
REAGENT BLANKING	NO
DIFFERENTIAL	NO
FILTERS (nm)	
FILTER 1	405
FILTER 2	NONE
TIMES (SEC)	
MIX 1	0.2
MIX 2	0.00
INCUBATION 1	40
INCUBATION 2	0
LAG PHASE	10
MEASURE	30
MEASUREMENT TYPE	
CALIBRATE	FACTOR*
FACTOR	3239
NORMAL RANGE	28 - 100