



Programação dos Reagentes CEPA

RA-XT 500 - 1000

Alfa Amilase Ceba

GAL G2 CNP – Cinético

CHEM #	*
NAME	ALFA AMILASE
IMMUNOASSAY	NO
TYPE	ZERO ORDER
INVERSE	NO
% SAMPLE	12(6µL)
FILTER P	405
BIC. CHEM	NO
DELAY TIME	00:30:00
BLANK	NO
%REAG. VOL	60(300µL)
2° REAG.	NO
UNITS	U/L
UNIT FACTOR	1
DECIMAL	0
RBL LOW	0,21
RBL RIGH	0,7
RANGE LOW	0
RANGE HIGH	1300
CAL FACTOR (**)	4703
REG. RATE	0
NORMAL LOW	0
NORMAL RIGH	60
SLOPE	1
INTERCEPTION	0
C1 *	0
C2 *	99999
D1 *	10
DELTA#	0.01