



Programação dos Reagentes CEPA

TARGA

Amilase Direta Cepa

GAL- G2 CNP- Cinético

TEST TYPE	KINETIC SINGLE
SERUM STARTER	INACTIVE
FILTERS A/B	405/700
UNITS	U/L
TEST METHOD	WHIT FACTOR
TEST METHODOLOGY	
NUMBER OF WASHES	1
DELAY TIME	0
INC REAGENT	60
READING TIME	90
TEST LIMIT (CONC)	1300
MAX ABS DELTA	400
REAGENT mABS Limit	800
REAGENT A/B	400
REACTION DIRECTION	INCREASING
REAGENT DILUTION	01:01:00
INITIAL ABS	1900
CURVE ACCEPTANCE	100,00%
AUTOMATIC PROFILE	(INACTIVE)
RERUN TEST RGT BLK H:M	00:00:00
SERUM PARAMETERS	
TEST'S NAMES	AMILASE DIRETO
SAMPLE VOLUME	8
DILUTION RATIO	1:2
MIN. MAX. M	0 – 60
MIN. MAX. F	0 – 60
MIN. MAX. B	