



Programação dos Reagentes CEPA

COBAS MIRA / MIRA PLUS

Bilirrubina Total Cepa

Sims-Horn-colorimétrico

Bilirrubina Total

- Main Reagent: Acelerador
- Start Reagent: Uma gota de nitrito de sódio para cada 1,5 mL de ácido sulfanílico

GENERAL		CALCULATION	
MEASUREMENT MODE	ABSORB	SAMPLE LIMIT	0,2
REACTION MODE	R-S - SR1	REACTION DIRECTION	INCREASE
CALIBRATION MODE	FACTOR	CHECK	OFF
REAGENT BLANK	REAG/DIL	CONVERSION FACTOR	1
CLEANER	BEFORE	OFF SET	0
WAVELENGTH	550	TEST RANGE LOW	0
DECIMAL POSITION	2 mg/dL	HIGH	15
UNIT	12 mg/dL	NORMAL RANGE LOW	NO
		HIGH	NO
ANALYSIS		NUMBER OF STEPS	1
POST DIL FACTOR	3	CALCULATION STEP A	ENDPOINT
CONC. FACTOR	NO	READING FIRST	2
SAMPLE CYCLE	1	LAST	14
VOLUME	10.4 µL	REACTION LIMIT	-
DILUTION NAME	H2O	CALIBRATION	
VOLUME	6,7 µL	CALIBRATION INTERVAL	ON REQUEST
REAGENT CYCLE	1	TIME	NO
VOLUME	150 µL	BLANK RANGE LOW	-0,01
START REAGENT 1 CYCLE	3	HIGH	0,36
VOLUME	16,6	REAGENTE RANGE LOW	-0,05
DIL	3,4	REAGENT RANGE HIGH	0,2