



Programação dos Reagentes CEPA

COBAS MIRA / MIRA PLUS

CK-NAC Cepa

Imunoinibição – Cinético UV

GENERAL		CALCULATION	
MEASUREMENT MODE	ABSORB	SAMPLE LIMIT / POINT	0,4 / T1
REACTION MODE	R-S	REACTION DIRECTION	INCREASE
CALIBRATION MODE	FACTOR	CHECK	OFF
REAGENT BLANK	REAG/DIL	CONVERSION FACTOR	1
CLEANER	NO	OFF SET	0
WAVELENGTH	340	TEST RANGE LOW	0
DECIMAL POSITION	0 U/L	HIGH	900
UNIT	21 U/L	NORMAL RANGE LOW	26
		HIGH	174
ANALYSIS		NUMBER OF STEPS	1
POST DIL FACTOR	5	CALCULATION STEP A	KINSEARCH
CONC. FACTOR	NO	READING FIRST	10
SAMPLE CYCLE	1	LAST	18
VOLUME	10 µL	REACTION LIMIT / POINT	0,9 / T1
DILUTION NAME	H2O	CALIBRATION	
VOLUME	10 µL	CALIBRATION INTERVAL	ON REQUEST
REAGENT CYCLE	1	TIME	NO
VOLUME	150 µL	BLANK RANGE LOW	-0,002
		HIGH	0,003
		REAGENTE RANGE LOW	0,100
		REAGENT RANGE HIGH	0,4
		FACTOR	4677