



Programação dos Reagentes CEPA

TARGA

CK-NAC Cepa

Imunoinibição-Cinético UV

TEST TYPE	KINETIC SINGLE
SERUM STARTER	INACTIVE
FILTERS A/B	340/700
UNITS	U/L
TEST METHOD	WHIT FACTOR
TEST METHODOLOGY	
NUMBER OF WASHES	1
DELAY TIME	0
INC REAGENT	180
READING TIME	120
TEST LIMIT (CONC)	900
MAX ABS DELTA	300
REAGENT mABS Limit	900
REAGENT A/B	300
REACTION DIRECTION	Increasing
REAGENT DILUTION	01:01:00
INITIAL ABS	1900
CURVE ACCEPTANCE	100,00%
AUTOMATIC PROFILE	(INACTIVE)
RERUN TEST RGT BLK H:M	00:00:00
SERUM PARAMETERS	
TEST'S NAMES	Creatino Kinase
SAMPLE VOLUME	12
DILUTION RATIO	1:4
MIN. MAX. M	24 – 195
MIN. MAX. F	24 – 170
MIN. MAX. B	