



Programação dos Reagentes CEPA

RA-XT 500 - 1000

Cálcio Cepa

Cresolftaleína complexona – Colorimétrico

CHEM #	(*)
NAME	CÁLCIO
IMMUNOASSAY	NO
TYPE	ENDPOINT
INVERSE	NO
% SAMPLE	14
FILTER P	600
BIC. CHEM	NO
DELAY TIME	2
BLANK	NO
%REAG. VOL	70 (350µL)
2° REAG.	NO
UNITS	mg/dL
UNIT FACTOR	1
DECIMAL	1
RBL LOW	0
RBL RIGH	0,5
RANGE LOW	0
RANGE HIGH	15
CAL FACTOR (**)	*
STANDARD VALUE	*
REG. RATE	0
NORMAL LOW	8,8
NORMAL RIGH	11
SLOPE	1
INTERCEPTION	0
END POINT LIMIT	0,01
C1 *	*
D1 *	*
DELTA#	*