



Programação dos Reagentes CEPA

COBAS MIRA / MIRA PLUS

Cálcio Cepa

Arsenazo – Colorimétrico

GENERAL		CALCULATION	
MEASUREMENT MODE	ABSORB	SAMPLE LIMIT	NO
REACTION MODE	R-S	REACTION DIRECTION	INCREASE
CALIBRATION MODE	CALIBRATOR	CHECK	ON
REAGENT BLANK	REAG/DIL	CONVERSION FACTOR	1
CLEANER	NO	OFF SET	0
WAVELENGTH	600	TEST RANGE LOW	0
DECIMAL POSITION	1 mg/dL	HIGH	18 g/L
UNIT	12 mg/dL	NORMAL RANGE LOW	9 g/L
		HIGH	10.7 g/L
ANALYSIS		NUMBER OF STEPS	1
POST DIL FACTOR	NO	CALCULATION STEP A	ENDPOINT
CONC. FACTOR	NO	READING FIRST	1
SAMPLE CYCLE	3	LAST	6
VOLUME	4 µL	REACTION LIMIT	-
DILUTION NAME	H2O	CALIBRATION	
VOLUME	10 µL	CALIBRATION INTERVAL	ON REQUEST
REAGENT CYCLE	1	TIME	NO
VOLUME	225 µL	BLANK RANGE LOW	-0,050
		HIGH	0,050
STANDARD POS	*	REAGENTE RANGE LOW	-
STD 1	*	REAGENT RANGE HIGH	-

*Inserido pelo usuário

Para Cobas Mira Plus, substituir CALIBRATOR por SLOPE AVG