



Programação dos Reagentes CEPA

COBAS MIRA / MIRA PLUS

Cálcio Cepa

Cresolftaleína complexona – Colorimétrico

GENERAL		CALCULATION	
MEASUREMENT MODE	ABSORB	SAMPLE LIMIT / POINT	NO
REACTION MODE	R-S-SR1	REACTION DIRECTION	INCREASE
CALIBRATION MODE	CALIBRATOR	CHECK	OF
REAGENT BLANK	REAG/DIL	CONVERSION FACTOR	1
CLEANER	NO	OFF SET	0
WAVELENGTH	550	TEST RANGE LOW	1
DECIMAL POSITION	1 mg/dL	HIGH	15 g/L
UNIT	12 mg/dL	NORMAL RANGE LOW	NO
		HIGH	NO
ANALYSIS		NUMBER OF STEPS	1
POST DIL FACTOR	3	CALCULATION STEP A	ENDPOINT
CONC. FACTOR	2	READING FIRST	2
SAMPLE CYCLE	3	LAST	4
VOLUME	3 µL	REACTION LIMIT	-
DILUTION NAME	H2O	CALIBRATION	
VOLUME	15 µL	CALIBRATION INTERVAL	EACH DAY
REAGENT CYCLE	1	TIME	4 Horas
VOLUME	100µL	BLANK RANGE LOW	NO
STAR R1 CYCLE	1	HIGH	NO
VOLUME	85µL	REAGENTE RANGE LOW	NO
DILUTION NAME	H2O	REAGENT RANGE HIGH	NO
VOLUME	10 µL		
STANDARD POS	*		-
STD 1	*		-

*Inserido pelo usuário

Para Cobas Mira Plus, substituir CALIBRATOR por SLOPE AVG