



Programação dos Reagentes CEPA

TARGA

Colesterol Total Cepas Enzimático Monoreagente

TEST TYPE	End Point Single
SERUM STARTER	INACTIVE
FILTERS A/B	510/700
UNITS	mg/dL
TEST METHOD	WHIT FACTOR
TEST METHODOLOGY	
NUMBER OF WASHES	1
DELAY TIME	0
INC REAGENT	300
READING TIME	10
TEST LIMIT (CONC)	1000
MAX ABS DELTA	0
REAGENT mABS Limit	200
REAGENT A/B	400
REACTION DIRECTION	Increasing
REAGENT DILUTION	01:01:00
INITIAL ABS	0
CURVE ACCEPTANCE	100,00%
AUTOMATIC PROFILE	(INACTIVE)
RERUN TEST RGT BLK H:M	00:00:00
SERUM PARAMETERS	
TEST'S NAMES	Colesterol Total
SAMPLE VOLUME	5
DILUTION RATIO	1:2
MIN. MAX. M	123 – 250
MIN. MAX. F	150 – 270
MIN. MAX. B	