



Colesterol Total Cepa

Enzimático Monoreagente

ANALYTICAL PARAMETERS

Analytical Conditions		Reanalysis conditions				Multi-Standards Setting			
R1 volume	80	Serum reac. Smp. Vol (µL)		3		Formula Points	Linear correc. 2	Axis conv.	No converte
R2 volume	0	Serum dilut. method. Vol (µL)		None					
R3 volume	0	Serum reac. Smp. Vol (d)		3			FV	MEAN	
R4 volume	0	Serum dilut. method. Vol (d)		None		BLK	0.00	...	
R1 diluent volume	0	Standards setting				1	*	---	
R2 diluent volume	0	BLK H		9.9999					
R3 diluent volume	0	BLK L		9.9999					
R4 diluent volume	0	STD H		9.9999					
Serum reac. s.volume	4	STD L		9.9999					
Serum diluent method	Standard	FV		9.9999					
		Abnml. (serum) H		9.9999					
Reaction Time	5 min.	Abnml. (serum) L		9.9999					
Reagent 1 stir.	Weak	Calculation method settings				Reation rate method			
Reagent 2 stir.	Weak	M-DET. P.I	0	Prozone		Cycle		3	
Reagent 3 stir.	Weak	M-DET. P.m	44	Prozone form	None	Factor		3.0	
Reagent 4 stir.	Weak	M-DET. P.n	46	Prozone limit	9999	Reac. Type		INC	
Sub-analy. conditions		M-DET. P.p	0	Prozone judge	Upper Limit	E2. corre.		Not dot	
Name	Colesterol	M-DET. P.r	0	Judge limit	9999	Blank (µL)		9.9999	
Digits	2			M-DET.P.m	0	Blank (d)		-9.999	
M-wave. L	505	Check D.P.I	0	M-DET.P.n	0	Sample (µL)		9.9999	
S-wave. L.	596	Limit value	0.003	S-DET. P.p	0	Sample (d)		-9.999	
Analy. mthd	EPA	Variance	10	S-DET.P.r	0	Endpoint method			
Calc. mtdh	STD					Re. Absorb (µL)		9.9999	
Qualit. judg	Not do					Re. Absorb (d)		-9.999	