



Programação dos Reagentes CEPA

Artax

Creatinina Cepa

Jaffé – Cinético de tempo fixo

PARAMETERS	CREA
METHOD	FIXED TIME
FILTER 1	510
FILTER 2	0
CUVETTE TEMP.	37
MAX. NORM. VAL.	1.2
MIN. NORM. VAL.	0.4
UNIT	mg/dL
STANDARD VAL.	*
FACTOR	0
# OF WASHES	4..
KIN. READ TIME	120
PRINTOUT GRAPH	N
TEST LIMIT	20
MAX. ABS. DELTA	250
REAGENT (A) μ L	500
REAGENT (B) μ L	0
SAMP. VOL. μ L	50
INC. REAG. (A)	60
INC. REAG. (B)	0
INC. TEMP.	37
STABILITY TIME	0
REACTION DIR.	INC.