



Programação dos Reagentes CEPA

COBAS MIRA / MIRA PLUS

Creatinina Cepa

Jaffé – Cinético de tempo fixo

GENERAL		CALCULATION	
MEASUREMENT MODE	ABSORB	SAMPLE LIMIT/POINT	0,36/T1
REACTION MODE	R-S	REACTION DIRECTION	INCREASE
CALIBRATION MODE	CALIBRATOR	CHECK	ON
REAGENT BLANK	REAG/DIL	CONVERSION FACTOR	1
CLEANER	BEFORE	OFF SET	0
WAVELENGTH	500	TEST RANGE LOW	0
DECIMAL POSITION	2 mg/dL	HIGH	10 mg/dL
UNIT	12 mg/dL	NORMAL RANGE LOW	0,4 mg/dL
		HIGH	1,3 mg/dL
ANALYSIS		NUMBER OF STEPS	1
POST DIL FACTOR	2	CALCULATION STEP A	ENDPOINT
CONC. FACTOR	NO	READING FIRST	2
SAMPLE CYCLE	1	LAST	7
VOLUME	20 µL	REACTION LIMIT	NO
DILUTION NAME	H2O	CALIBRATION	
VOLUME	30 µL	CALIBRATION INTERVAL	ON REQUEST
REAGENT CYCLE	1	TIME	NO
VOLUME	200 µL	BLANK RANGE LOW	NO
		HIGH	NO
STANDARD POS	*	REAGENTE RANGE LOW	NO
STD 1	*	REAGENT RANGE HIGH	NO

*Inserido pelo usuário

Para Cobas Mira Plus, substituir CALIBRATOR por SLOPE AVG