



Programação dos Reagentes CEPA

RA-XT 500 - 1000

Creatinina Cepa

Jaffé – Cinético de tempo fixo

CHEM #	*
NAME	CREATININA
IMMUNOASSAY	NO
TYPE	FIRST ORDER
INVERSE	NO
% SAMPLE	36
FILTER P	500
BIC. CHEM	NO
DELAY TIME	00:30:00
INCUBATE	1
%REAG. VOL	72
2° REAG.	NO
UNITS	mg/dL
UNIT FACTOR	1
DECIMAL	0
RBL LOW	0
RBL RIGH	0,5
RANGE LOW	0
RANGE HIGH	10
CAL FACTOR (**)	1
STANDARD VALUE	*
REG. RATE	0
NORMAL LOW	0,4
NORMAL RIGH	1,4
SLOPE	1
INTERCEPTION	0
LINEAR FACTOR	2,6
FIRST LIMIT	0,1
C1 *	*
1 ACCEPT	1