

Creatinina Cepa

Jaffé – Cinético de tempo fixo

R1 – Tampão
R3 – Ác. Pícrico

ANALYTICAL PARAMETERS

Analytical Conditions		Reanalysis conditions		Multi-Standards Setting			
R1 volume	120	Serum reac. Smp. Vol (µL)	3	Formula Points	Linear correc. 2	Axis conv.	No converte
R2 volume	0	Serum dilut. method. Vol (µL)	None				
R3 volume	40	Serum reac. Smp. Vol (d)	3		FV	MEAN	
R4 volume	0	Serum dilut. method. Vol (d)	None	BLK	0.00	...	
R1 diluent volume	0	Standards setting		1	*	---	
R2 diluent volume	0	BLK H	9.9999				
R3 diluent volume	0	BLK L	9.9999				
R4 diluent volume	0	STD H	9.9999				
Serum reac. s.volume	25	STD L	9.9999				
Serum diluent method	Standard	FV	9.9999				
		Abnml. (serum) H	9.9999				
Reaction Time	5 min.	Abnml. (serum) L	9.9999				
Reagent 1 stir.	Weak	Calculation method settings		Reation rate method			
Reagent 2 stir.	Weak	M-DET. P.l	0 Prozone	Cycle		3	
Reagent 3 stir.	Weak	M-DET. P.m	96 Prozone form	Factor		3.0	
Reagent 4 stir.	Weak	M-DET. P.n	98 Prozone limit	Reac. Type		INC	
Sub-analy. conditions		M-DET. P.p	0 Prozone judge	E2. corre.		Not dot	
Name	Crea	M-DET. P.r	0 Judge limit	Blank (µL)		9.9999	
Digits	2		M-DET.P.m	Blank (d)		-9.999	
M-wave. L	505	Check D.P.l	0 M-DET.P.n	Sample (µL)		9.9999	
S-wave. L.	571	Limit value	0.003 S-DET. P.p	Sample (d)		-9.999	
Analy. mthd	EPA	Variance	10 S-DET.P.r				
Calc. mtdh	STD			Endpoint method			
Qualit. judg	Not do			Re. Absorb (µL)		9.9999	
				Re. Absorb (d)		-9.999	