

Fator Reumatóide

Turbidimetria

Name	FR	Prozone check	No
Abbr. Name	*		
Mode	Two Point	Ref. male low	0 IU/L
		Ref. male high	30 IU/L
Wavelength	620 nm	Ref. female low	0 IU/L
Units	IU/L	Ref. female high	30 IU/L
Decimals	1		
Low Conc.	0 IU/L		
High Conc.	150 U/L		
Calibrator Name			
Repeat	2		
Number	5		
Concentration			
Interval	*		
Cut-off	No	Correlat. factor	1.000
		Correlat. offset	0.000
DUAL MODE			
Name	FR		
Sample blank	No	Low Absorbance	- 0.050
R1 bottle	25 mL	High Absorbance	3.000
normal volume	350 µL	R. Abs. L. Limit	- 0.050
rerun volume	270 µL	R. Abs. H. Limit	3.000
Sample		R.Abs.Deviation	2.000
normal volume	3 µL	Reagent Blank	Yes §
rerun volume	2 µL	Cal Low Limit	*
R2 bottle	25 mL	Cal. High Limit	*
normal volume	40 µL	Factor	*
rerun volume	30 µL		
Predilution	No		
Slope Blank	No		
Delay, min. time	-3, 150 sec.		