



Programação dos Reagentes CEPA

TARGA

Fator Reumatóide TURB

Turbidimetria

TEST TYPE	FIXED TIME SINGLE
SERUM STARTER	INACTIVE
FILTERS A/B	630
UNITS	U/mL
TEST METHOD	CUBIC SPLINE
TEST METHODOLOGY	4
NUMBER OF WASHES	1
DELAY TIME	0
INC REAGENT	60/15
READING TIME	120
TEST LIMIT (CONC)	160
MAX ABS DELTA	300
REAGENT mABS Limit	1400
REAGENT A/B	360 /40
REACTION DIRECTION	INCREASING
REAGENT DILUTION	1:1
INITIAL ABS	1999
CURVE ACCEPTANCE	100,00%
AUTOMATIC PROFILE	(INACTIVE)
RERUN TEST RGT BLK H:M	00:00:00
SERUM PARAMETERS	
TEST'S NAMES	FATOR REUMATÓIDE
SAMPLE VOLUME	4
DILUTION RATIO	1:2
MIN. MAX. M	0.0 - 20
MIN. MAX. F	0.0 - 20
MIN. MAX. B	