

Fosfatase Alcalina Cepa

4-nitrofenilfosfato – Tampão AMP (IFCC) – Cinético

TEST TYPE	Kinetic Single
SERUM STARTER	INACTIVE
FILTERS A/B	405/ 700
UNITS	U/L
TEST METHOD	WHIT FACTOR
TEST METHODOLOGY	
NUMBER OF WASHES	1
DELAY TIME	0
INC REAGENT	60
READING TIME	90
TEST LIMIT (CONC)	1200
MAX ABS DELTA	250
REAGENT mABS Limit	900
REAGENT A/B	400
REACTION DIRECTION	Increasing
REAGENT DILUTION	01:01:00
INITIAL ABS	1990
CURVE ACCEPTANCE	100,00%
AUTOMATIC PROFILE	(INACTIVE)
RERUN TEST RGT BLK H:M	00:00:00
SERUM PARAMETERS	
TEST'S NAMES	Fosfatase Alcalina
SAMPLE VOLUME	8
DILUTION RATIO	1:4
MIN. MAX. M	26 – 117
MIN. MAX. F	26 – 117
MIN. MAX. B	