



Programação dos Reagentes CEPA

Artax

Gama GT Cepa

IFCC – Cinético

PARAMETERS	GGT
METHOD	KINETIC
FILTER 1	405
FILTER 2	0
CUVETTE TEMP.	37
MAX. NORM. VAL.	60
MIN. NORM. VAL.	10
UNIT	U/L
STANDARD VAL.	0
FACTOR	1111
# OF WASHES	2..
KIN. READ TIME	60
PRINTOUT GRAPH	N
TEST LIMIT	300
MAX. ABS. DELTA	250
REAGENT (A) μ L	500
REAGENT (B) μ L	0
SAMP. VOL. μ L	50
INC. REAG. (A)	60
INC. REAG. (B)	0
INC. TEMP.	37
STABILITY TIME	0
REACTION DIR.	INC.