

TEST TYPE	Kinetic Single
SERUM STARTER	INACTIVE
FILTERS A/B	405/ 700
UNITS	U/L
TEST METHOD	WHIT FACTOR
TEST METHODOLOGY	
NUMBER OF WASHES	1
DELAY TIME	0
INC REAGENT	60
READING TIME	90
TEST LIMIT (CONC)	300
MAX ABS DELTA	250
REAGENT mABS Limit	1100
REAGENT A/B	400
REACTION DIRECTION	Increasing
REAGENT DILUTION	01:01:00
INITIAL ABS	1300
CURVE ACCEPTANCE	100,00%
AUTOMATIC PROFILE	(INACTIVE)
RERUN TEST RGT BLK H:M	00:00:00
SERUM PARAMETERS	
TEST'S NAMES	Gama GT
SAMPLE VOLUME	40
DILUTION RATIO	1:4
MIN. MAX. M	15 – 86
MIN. MAX. F	10 – 40
MIN. MAX. B	