



Programação dos Reagentes CEPA

ABBOTT VP®

Glicose Cepa

Enzimático Monoreagente

NAME INDEX – TEST NAME	10 – (COL)	LOW STANDARDS	100
TEMPERATURE	37°C	HIGH STANDARDS	100
FILTER CODE	13 (500/600)	REAG. BLANK	YES
UNITS	05 (mg/dL)	END POINT	YES
DILUTION 1:	101	BGN PRT REVOLUTION	3
REV. TIME	2	PRT REVOLUTION	1
AUX. DISP	NO	INITIAL REAGENT	0,1
FRR	YES	UP LIMIT	YES
REACTION UP	YES	MAX ABS. LIMIT	0,9
STANDARDS	YES		