



Programação dos Reagentes CEPA

RA-XT 500 - 1000

Glicose Ceba

Enzimático Monoreagente

CHEM #	(*)
NAME	GLICOSE
IMMUNOASSAY	NO
TYPE	ENDPOINT
INVERSE	NO
% SAMPLE	7
FILTER P	500
BIC. CHEM	NO
DELAY TIME	5
BLANK	NO
%REAG. VOL	70 (350µL)
2° REAG.	NO
UNITS	mg/dL
UNIT FACTOR	1
DECIMAL	1
RBL LOW	0
RBL HIGH	0,04
RANGE LOW	0
RANGE HIGH	500
CAL FACTOR (**)	*
STANDARD VALUE	*
REG. RATE	0
NORMAL LOW	76
NORMAL HIGH	110
SLOPE	1
INTERCEPTION	0
END POINT LIMIT	0,01
C1 *	*
D1 *	*
DELTA#	*