



Programação dos Reagentes CEPA

TARGA

Colesterol HDL Cepa

Seleção Direta

TEST TYPE	End Point 2 Point With Startec
SERUM STARTER	INACTIVE
FILTERS A/B	510
UNITS	mg/dL
TEST METHOD	WHIT FACTOR
TEST METHODOLOGY	
NUMBER OF WASHES	1
DELAY TIME	0
INC REAGENT	150 . 250
READING TIME	20
TEST LIMIT (CONC)	150
MAX ABS DELTA	0
REAGENT mABS Limit	300
REAGENT A/B	300 100
REACTION DIRECTION	Increasing
REAGENT DILUTION	01:01:00
INITIAL ABS	1900
CURVE ACCEPTANCE	100,00%
AUTOMATIC PROFILE	(INACTIVE)
RERUN TEST RGT BLK H:M	00:00:00
SERUM PARAMETERS	
TEST'S NAMES	HDL Direto
SAMPLE VOLUME	3
DILUTION RATIO	1:1
MIN. MAX. M	35 – 70
MIN. MAX. F	35 – 70
MIN. MAX. B	