



Colesterol HDL Cepas

Precipitação

ANALYTICAL PARAMETERS

Analytical Conditions		Reanalysis conditions				Multi-Standards Setting			
R1 volume	80	Serum reac. Smp. Vol (μL)		3		Formula Points	Linear correc. 2	Axis conv.	No converte
R2 volume	0	Serum dilut. method. Vol (μL)		None					
R3 volume	0	Serum reac. Smp. Vol (d)		3			FV	MEAN	
R4 volume	0	Serum dilut. method. Vol (d)		None		BLK	0.00	...	
R1 diluent volume	0	Standards setting				1	*	---	
R2 diluent volume	0	BLK H		9.9999					
R3 diluent volume	0	BLK L		9.9999					
R4 diluent volume	0	STD H		9.9999					
Serum reac. s.volume	20	STD L		9.9999					
Serum diluent method	Standard	FV		9.9999					
		Abnml. (serum) H		9.9999					
Reaction Time	5 min.	Abnml. (serum) L		9.9999					
Reagent 1 stir.	Weak	Calculation method settings				Reation rate method			
Reagent 2 stir.	Weak	M-DET. P.I	0	Prozone		Cycle	3		
Reagent 3 stir.	Weak	M-DET. P.m	44	Prozone form	None	Factor	3.0		
Reagent 4 stir.	Weak	M-DET. P.n	46	Prozone limit	9999	Reac. Type	INC		
Sub-analy. conditions		M-DET. P.p	0	Prozone judge	Upper Limit	E2. corre.	Not dot		
Name	HDL PRE.	M-DET. P.r	0	Judge limit	9999	Blank (μL)	9.9999		
Digits	0			M-DET.P.m	0	Blank (d)	-9.999		
M-wave. L	505	Check D.P.I	0	M-DET.P.n	0	Sample (μL)	9.9999		
S-wave. L.	596	Limit value	0.003	S-DET. P.p	0	Sample (d)	-9.999		
Analy. mthd	EPA	Variance	10	S-DET.P.r	0	Endpoint method			
Calc. mtdh	STD					Re. Absorb (μL)	9.9999		
Qualit. judg	Not do					Re. Absorb (d)	-9.999		