

PROGRAMAÇÃO SUGERIDA PARA LABMAX 240® e PRESTIGE24i®



PCR TURB CEPA

CALIBRAÇÃO

- ✓ Padrão incluso no kit.
- ✓ Calibrar com 2 pontos, utilizando salina 0,9% para o **Padrão de 0 mg/dL** e o **Padrão do Kit**
- ✓ **R1** = Tampão | **R2** – Látex

Test	CRP
Unit	mg/dL
Decimals	1
Analysis type	Two point end assay 1
Main wavelength	340
Sub wave length	-
Method	Turbidimetry
Correction	1.00x + 0
Calibration type	Point to Point
Standard	Multipoint calibration
Normal range	*
Sample	16 µL
Automatic dilution	NO
R1	250 µL
R2	50 µL
Read Main	54
Sub	31
Prozone check	NO

* Inserido pelo usuario