



Programação dos Reagentes CEPA

RA-XT 500 - 1000

LDH Ceba

Piruvato – Cinético UV

CHEM #	*
NAME	LDH
IMMUNOASSAY	NO
TYPE	ZERO ORDER
INVERSE	NO
% SAMPLE	14(7µL)
FILTER P	340
BIC. CHEM	NO
DEPLETION LIMIT	0,15
DELAY TIME	00:30:00
BLANK	NO
%REAG. VOL	66 (330µL)
2º REAG.	NO
UNITS	U/L
UNIT FACTOR	1
DECIMAL	0
RBL LOW	0
RBL HIGH	0,7
RANGE LOW	0
RANGE HIGH	1000
CAL FACTOR (**)	13833
REG. RATE	0
NORMAL LOW	123
NORMAL HIGH	228
SLOPE	1
INTERCEPTION	0
C1 *	0
C2 *	99999
D1 *	10
DELTA#	0,01