



Programação dos Reagentes CEPA

RA-XT 500 - 1000

Magnésio Ceba

Calmagita – Colorimétrico

CHEM #	(*)
NAME	MG
IMMUNOASSAY	NO
TYPE	ENDPOINT
INVERSE	NO
% SAMPLE	6 (3µL)
FILTER P	500
BIC. CHEM	NO
DELAY TIME	1
BLANK	NO
%REAG. VOL	60 (300µL)
2° REAG.	NO
UNITS	mg/dL
UNIT FACTOR	1
DECIMAL	1
RBL LOW	0,28
RBL RIGH	0,49
CAL FACTOR (**)	*
STANDARD VALUE	*
REG. RATE	0
NORMAL LOW	1,8
NORMAL RIGH	2,1
SLOPE	1
INTERCEPTION	0
END POINT LIMIT	0,03
C1 *	*
D1 *	*
DELTA#	*