

PROGRAMAÇÃO SUGERIDA PARA Mindraw BS120_240®



PCR TURB CEPA

CALIBRAÇÃO

- ✓ Padrão incluso no kit.
- ✓ Utilizar salina 0,9% para o **Padrão de 0 mg/mL**
- ✓ **Reagente A** = Tampão | **Reagente B** – Látex

Parameters							
No.	*	Assay	PCR	Type	Endpoint	Reaction	Ascending
Unit	mg/dL	Precision	0	Prim. Wave.	340	Sec. Wave.	0
Sample Vol.	16	R1 Vol.	250	R2 Vol.	50	Lin. Limit	20
Inc. Time	10	Antigen		Response		S. Volume	0
R. Time	-1 / 20	Substrate	0	L Limit	0	Ratio	0
				U Limit	0		
R1 Blank		Mix. R Blank		Linearity		Full Name	PCR
L Limit	0	L Limit	0	L Limit	0	Print No.	1
U Limit	0	U Limit	0	U Limit	0		
Ref. Interval				Calibration			
Low	*			Cal. Type	Linear		
High	*			Calibrator	Um ponto		