



Programação dos Reagentes CEPA

TARGA

Proteína C-Reativa TURB

Turbidimetria

TEST TYPE	FIXED TIME SINGLE
SERUM STARTER	INACTIVE
FILTERS A/B	546
UNITS	mg/L
TEST METHOD	WHIT FACTOR
TEST METHODOLOGY	4
NUMBER OF WASHES	1
DELAY TIME	0
INC REAGENT	60/15
READING TIME	120
TEST LIMIT (CONC)	150
MAX ABS DELTA	500
REAGENT mABS Limit	1300
REAGENT A/B	360 /40
REACTION DIRECTION	INCREASING
REAGENT DILUTION	01:01:00
INITIAL ABS	1999
CURVE ACCEPTANCE	100,00%
AUTOMATIC PROFILE	(INACTIVE)
RERUN TEST RGT BLK H:M	00:00:00
SERUM PARAMETERS	
TEST'S NAMES	C-REACTIVE PROTEIN
SAMPLE VOLUME	3
DILUTION RATIO	1:1
MIN. MAX. M	0.0 - 6.0
MIN. MAX. F	0.0 - 6.0
MIN. MAX. B	