



Programação dos Reagentes CEPA

COBAS MIRA / MIRA PLUS

Proteínas Totais Cepa

Biureto – Colorimétrico

GENERAL		CALCULATION	
MEASUREMENT MODE	ABSORB	SAMPLE LIMIT	NO
REACTION MODE	R-S	REACTION DIRECTION	INCREASE
CALIBRATION MODE	CALIBRATOR	CHECK	ON
REAGENT BLANK	REAG/DIL	CONVERSION FACTOR	1
CLEANER	NO	OFF SET	0
WAVELENGTH	550	TEST RANGE LOW	0
DECIMAL POSITION	2	HIGH	12
UNIT	g/dL	NORMAL RANGE LOW	NO
		HIGH	NO
ANALYSIS		NUMBER OF STEPS	1
POST DIL FACTOR	2	CALCULATION STEP A	ENDPOINT
CONC. FACTOR	NO	READING FIRST	CB
SAMPLE CYCLE	2	LAST	6
VOLUME	6 µL	REACTION LIMIT	-
DILUTION NAME	H2O	CALIBRATION	
VOLUME	24 µL	CALIBRATION INTERVAL	ON REQUEST
REAGENT CYCLE	1	TIME	NO
VOLUME	275 µL	BLANK RANGE LOW	-0,1 A
		HIGH	0,2 A
STANDARD POS	*	REAGENTE RANGE LOW	-0,1 A
STD 1	*	REAGENT RANGE HIGH	0,2 A

*Inserido pelo usuário

Para Cobas Mira Plus, substituir CALIBRATOR por SLOPE AVG