

Uréia Cinética Cepa

Urease/GluDH -Cinético UV

TEST TYPE	Fixed time Single
SERUM STARTER	INACTIVE
FILTERS A/B	340 / 700
UNITS	mg/dL
TEST METHOD	WHIT FACTOR
TEST METHODOLOGY	
NUMBER OF WASHES	1
DELAY TIME	0
INC REAGENT	30
READING TIME	40
TEST LIMIT (CONC)	300
MAX ABS DELTA	400
REAGENT mABS Limit	900
REAGENT A/B	400
REACTION DIRECTION	Decreasing
REAGENT DILUTION	01:01:00
INITIAL ABS	1999
CURVE ACCEPTANCE	100,00%
AUTOMATIC PROFILE	(INACTIVE)
RERUN TEST RGT BLK H:M	00:00:00
SERUM PARAMETERS	
TEST'S NAMES	Ureia UV
SAMPLE VOLUME	4
DILUTION RATIO	1:2
MIN. MAX. M	10 – 50
MIN. MAX. F	10 – 50
MIN. MAX. B	