



Programação dos Reagentes CEPA

COBAS MIRA / MIRA PLUS

Uréia Urease Cepa

Urease Indofenol – Colorimétrico

GENERAL		CALCULATION	
MEASUREMENT MODE	ABSORB	SAMPLE LIMIT	NO
REACTION MODE	R-S- SR1	REACTION DIRECTION	INCREASE
CALIBRATION MODE	CALIBRATOR	CHECK	ON
REAGENT BLANK	REAG/DIL	CONVERSION FACTOR	1
CLEANER	NO	OFF SET	0
WAVELENGTH	600	TEST RANGE LOW	0
DECIMAL POSITION	1 mg/dL	HIGH	300
UNIT	12 mg/dL	NORMAL RANGE LOW	10
		HIGH	50
ANALYSIS		NUMBER OF STEPS	1
POST DIL FACTOR	NO	CALCULATION STEP A	ENDPOINT
CONC. FACTOR	NO	READING FIRST	3
SAMPLE CYCLE	2	LAST	24
VOLUME	2 µL	REACTION LIMIT	-
DILUTION NAME	H2O	CALIBRATION	
VOLUME	10 µL	CALIBRATION INTERVAL	ON REQUEST
REAGENT CYCLE	1	TIME	NO
VOLUME	100 µL	BLANK RANGE LOW	NO
START R1 CYCLE	12	HIGH	NO
VOLUME	85µL	REAGENTE RANGE LOW	NO
DILUTION NAME	H2O	REAGENT RANGE HIGH	NO
VOLUME	10µL		
STANDARD POS	*		
STD 1	*		

*Inserido pelo usuário

Para Cobas Mira Plus, substituir CALIBRATOR por SLOPE AVG