



Programação dos Reagentes CEPA

COBAS MIRA / MIRA PLUS

Ácido Úrico Cepa

Enzimático - Colorimétrico

GENERAL		CALCULATION	
MEASUREMENT MODE	ABSORB	SAMPLE LIMIT	NO
REACTION MODE	R-S	REACTION DIRECTION	INCREASE
CALIBRATION MODE	CALIBRATOR	CHECK	ON
REAGENT BLANK	REAG/DIL	CONVERSION FACTOR	1
CLEANER	NO	OFF SET	0
WAVELENGTH	500	TEST RANGE LOW	0
DECIMAL POSITION	1 mg/dL	HIGH	25
UNIT	12 mg/dL	NORMAL RANGE LOW	2,2
		HIGH	7,2
ANALYSIS		NUMBER OF STEPS	1
POST DIL FACTOR	3	CALCULATION STEP A	ENDPOINT
CONC. FACTOR	NO	READING FIRST	1
SAMPLE CYCLE	2	LAST	13
VOLUME	5 µL	REACTION LIMIT	-
DILUTION NAME	H2O	CALIBRATION	
VOLUME	30 µL	CALIBRATION INTERVAL	ON REQUEST
REAGENT CYCLE	1	TIME	NO
VOLUME	200 µL	BLANK RANGE LOW	-0.20
		BLANK RANGE HIGH	0.20
STANDARD POS	*	REAGENTE RANGE LOW	NO
STD 1	*	REAGENT RANGE HIGH	NO
*Inserido pelo usuário			

*Inserido pelo usuário

Para Cobas Mira Plus, substituir CALIBRATOR por SLOPE AVG